YOUR SAMPLE WILL NOT BE PROCESSED UNLESS IT IS REGISTERED

Before Saliva Collection

1. DO NOT EAT, DRINK, CHEW, GUM, BRUSH TEETH, USE MOUTHWASH OR SMOKE **FOR AT LEAST 30 MINUTES** BEFORE SALIVA COLLECTION.
2. Relax your cheeks and gently massage them with your fingers for 15 - 30 seconds. This helps to produce more reliable result.

EPIAGE SALIVA COLLECTION KIT, 3 ML

INSTRUCTIONS FOR USE
typically takes 2-3 minutes to complete

WARNING:

- Do not ingest the liquid in this kit.
- Avoid eye contact.
- Keep out of reach from children.

1. Carefully unscrew the cap from the tube containing the stabilizing buffer and attach the funnel to it.

2. Spit saliva (not phlegm) into the funnel that you have now attached to the collection tube until the volume reaches the 5 ml line.
Important: Over-filling the tube during saliva collection may render failure of the test.

3. Carefully unscrew the funnel to prevent spillage of the collected saliva from the tube. The funnel can be discarded.

4. Use the supplied cap to close the collection tube tightly. Invert and shake the capped tube 10 - 15 times to ensure the saliva and the stabilizer buffer are fully mixed.

5. Put the tube into the ziplock bag.

6. Please **keep this instruction sheet** as you will need **your barcode number** to access your results.

7. Ship to us using the enclosed envelope.

8. Register this kit on our "EpiAge" app. **YOUR SAMPLE WILL NOT BE PROCESSED UNLESS IT IS REGISTERED.**

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